
endolite

Esprit

ESP22L1 - ESP30R8
ESP22L1D - ESP30R8D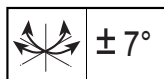

endolite

Available with separately supplied alignment devices
Dispositifs d'alignement disponibles séparément
erhältlich mit zusätzlich bestellbarer Montageklammer und Schneidehilfe
Disponibile con diversi attacchi prossimali (venduti separatamente)
Disponible con dispositivos de alineación suministrados por separado

Instructions for Use
Instructions D'Utilisation
Gebrauchsanweisung
Istruzioni per L'Uso
Instrucciones de Us

EN	FR	DE	IT	ES
2	12	22	32	42

1 Description and Purpose

Application:

These instructions are for use by the Practitioner.

The Esprit is to be used exclusively as part of a lower limb prosthesis.

A moderate energy return foot. Independent heel and toe spring provides some axial deflection. The split toe provides good ground compliance.

This device is recommended for users that have the potential to achieve Activity Level 3.

Of course there are exceptions and in our recommendation we want to allow for unique, individual circumstances. There may also be a number of users in the Activity Levels 2 and 4* who would benefit from the Esprit, but this decision should be made with sound and thorough justification.

*(maximum user weight 100kg and always use one higher spring rate category than shown in the Spring Set Selection table).

Appropriate footwear must be worn to avoid the risk of slipping in wet environments

Contra-indications:

This device may not be suitable for Activity Level 1 individuals or for competitive sports events, as these types of users will be better served by a specially designed prosthesis optimized for their need.

Intended for single user.

Ensure that the user has understood all instructions for use, drawing particular attention to the section regarding Maintenance.

Spring Set Selection

		User Weight								
Activity		44-52	53-59	60-68	69-77	78-88	89-100	101-116	117-125	kg
		(100-115)	(116-130)	(131-150)	(151-170)	(171-195)	(196-220)	(221-255)	(256-275)	(lbs)
3		1	2	3	4	5	6	7	8	Foot Spring set

Activity Level 3

Has the ability or potential for ambulation with variable cadence. Typical of the community ambulator who has the ability to traverse most environmental barriers and may have vocational, therapeutic, or exercise activity that demands prosthetic utilization beyond simple locomotion.

Note:

If in doubt choosing between two categories, choose the higher rate spring set.

Foot Spring set recommendations shown are for transtibial users.

For transfemoral users we suggest selecting a spring set one category lower, refer to fitting advice Section 7 to ensure satisfactory function and range of movement

Order Example:


ESP 25L 3 e.g. ESP25L3
Size Spring Set
Side Category

Available from size 22 to size 30:
ESP22L1 to ESP30R8
ESP22L1D to ESP30R8D
(add 'D' for a dark tone foot shell)

2 Construction

Principal parts:

- Carrier Assembly (Aluminium/ Titanium/St. Stl.)
- Heel & Toe Springs (e-Carbon)
- Spring attachment screws (Titanium/St. Stl.)
- Glide Sock (UHM PE)
- Foot shell (PU)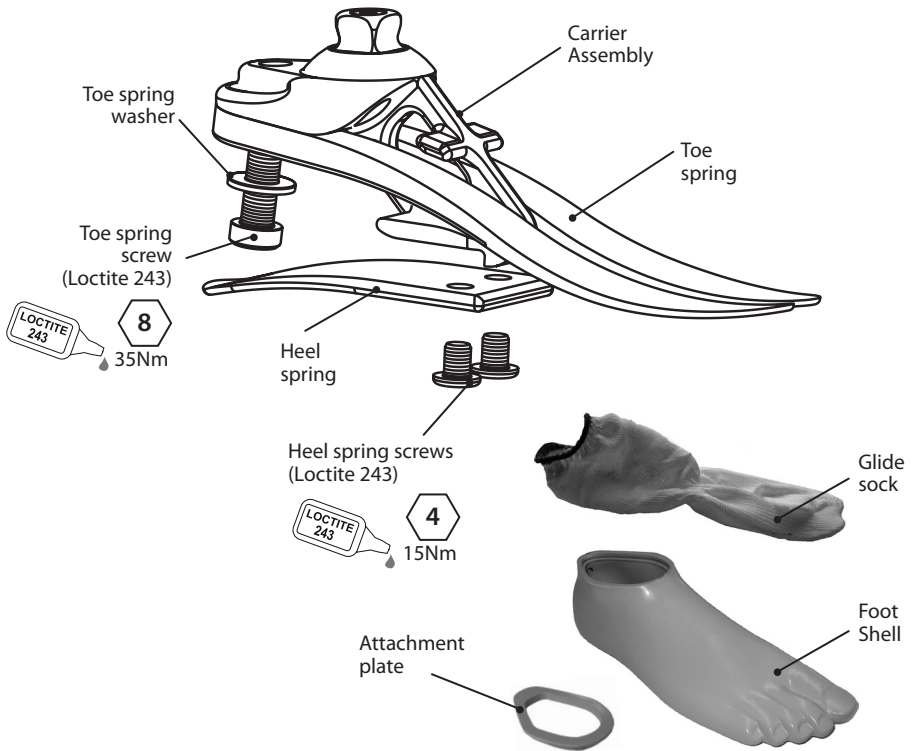

3 Function

The Esprit comprises an e-carbon toe and independent heel spring. Heel and toe springs are attached to the carrier using titanium and Stainless Steel screws. The foot is wrapped in a UHM PE sock which is in turn surrounded by a PU foot shell.

4 Maintenance

Maintenance must be carried out by competent personnel.

It is recommended that the following maintenance is carried out annually:

- Remove the foot shell and glide sock, check for damage or wear and replace if necessary.
- Check all screws for tightness, clean and reassemble as necessary.
- Visually check the heel and toe springs for signs of delamination or wear and replace if necessary. Some surface damage may occur after a period of use, this does not affect the function or strength of the foot.

The wearer must be handed the user information card supplied, and be advised of the following:

- Any changes in performance of this device must be reported to the practitioner e.g. reduced energy return or unusual noises.
- The practitioner must also be informed of any changes in body weight and/or activity level.

Cleaning:

Use a damp cloth and mild soap to clean outside surfaces, do not use aggressive cleansers.

5 Limitations on use:

Intended life:

A local risk assessment should be carried out based upon activity and usage.

Lifting loads:

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

Environment:

The product is water proof to a maximum depth of 1 metre.

Thoroughly rinse with fresh water after use in abrasive environments such as those that may contain sand or grit, for example, to prevent wear or damage to moving parts.


Thoroughly rinse with fresh water after use in salt or chlorinated water.

Foot products must be adequately finished to prevent water ingress into the foot shell where possible. If water enters the foot shell, the limb should be inverted and dried before further use.

Exclusively for use between -15°C and 50°C (5°F to 122°F).

It is recommended that only Endolite products be used in conjunction with the Esprit.

6 Bench Alignment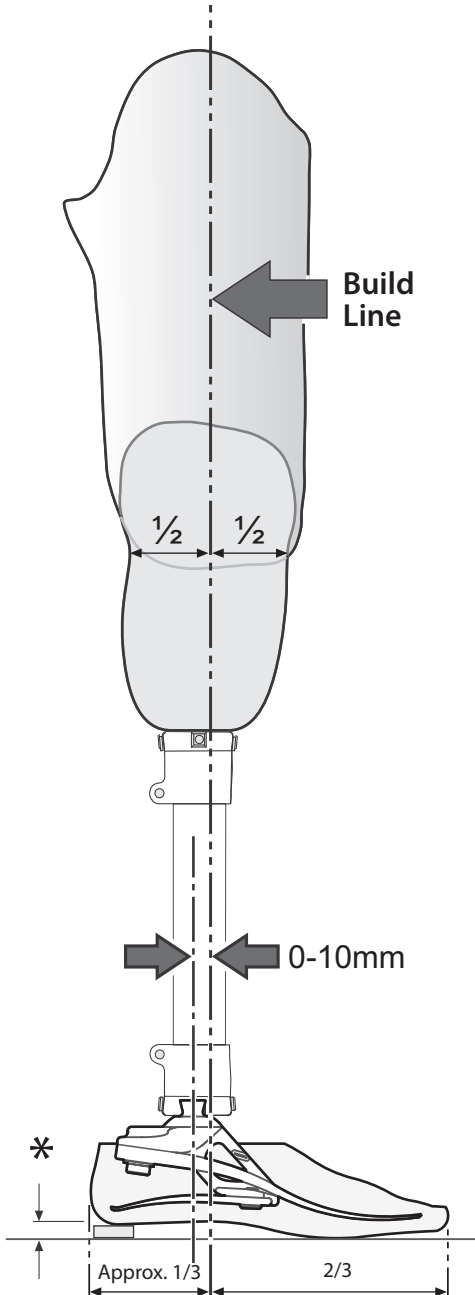

Static alignment

Set Up length

With flexion, adduction and abduction properly accommodated, set the limb length 5mm longer than the sound side to allow for compression and deflection of the foot springs during gait. This should be re-assessed once the dynamic trial commences and the length adjusted accordingly.

Load Line

This should fall between the centre line of the pyramid and 10mm anterior (with heel height properly accommodated). The socket should be positioned accordingly.

Dynamic alignment

Coronal Plane

Ensure that M-L thrust is minimal by adjusting relative positions of socket and foot.

Sagittal Plane

Check for smooth transition from heel strike to toe off. Ensure also that when standing the heel and toe are evenly loaded and that both are touching the floor.

Ankle alignment note:

When using the cosmesis attachment plate with Endolite Pyramid adaptors, it may be necessary to replace the medial and lateral grub screws with the shortened items supplied to avoid any chafing of the screws against the plate.

If high levels of inversion / eversion are required, the adaptor should be refitted with the original screws to ensure maximum thread engagement.

Trans-femoral Alignment

Align trans-femoral devices according to fitting instructions supplied with the knee, keeping the build line relative to the Esprit as shown.

* Allow for users own footwear

7 Fitting Advice

Spring sets are supplied as matched pairs i.e. the heel and toe spring are designed to work together to give smooth progression for most users.

Heel stiffness

Progression throughout the stance phase should be smooth; heel function is key to this process:


- Too soft a heel or load line excessively posterior will result in sinking at heel strike and difficulty in getting over the toe.
- Too hard a heel or load line excessively anterior will result in a rapid progression through mid-stance or jarring at heel strike.

	<i>Symptoms</i>	<i>Remedy</i>
Heel too soft	<ul style="list-style-type: none">• Sinking at heel strike• Difficulty climbing over the toe (toe feels too hard)	<ol style="list-style-type: none">1. Move socket anteriorly in relation to the foot (excess movement may result in drop off)2. Fit a stiffer spring set
Heel too hard	<ul style="list-style-type: none">• Rapid transition from heel strike through stance phase• Difficulty in controlling heel action, foot jars into mid-stance• Foot feels too rigid	<ol style="list-style-type: none">1. Move socket posteriorly in relation to foot2. Fit softer spring set

Please contact your supplier if it is not possible to achieve a smooth gait after following the advice above.

8 Assembly

1 Assemble toe spring onto carrier, use Loctite 243 (926012) and torque to 35Nm.

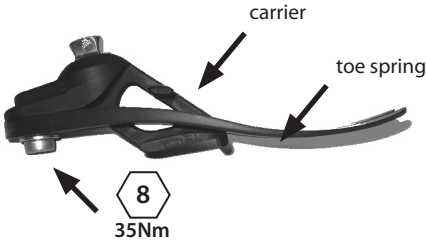2

heel spring

4
15Nm

Assemble heel spring onto carrier, use Loctite 243 (926012) and torque to 15Nm.

3

8


Cover appropriate lines on carrier with permanent black marker to leave spring set number showing

4

928017
Lubricate toe and heel if required. (Footshell is pre-lubricated)

Fit glide sock over the carrier assembly as shown.


5 Slide carrier/heel spring assembly into the foot shell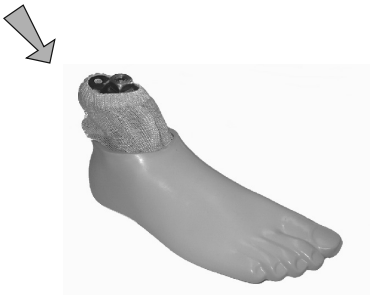

6

toe spring location in foot shell


8 Assembly *(continued)*

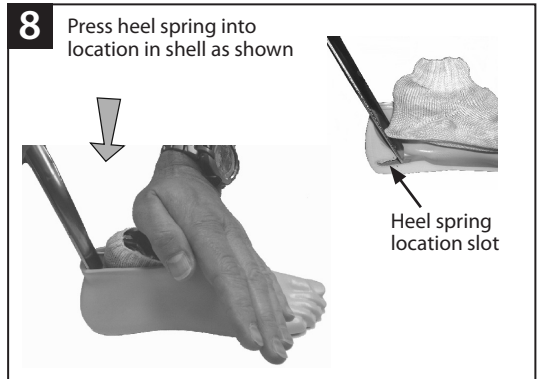9 Fit attachment plate into groove around foot shell aperture. Ensure glide sock does not get trapped when assembling to female pyramid part.

10 Bond foam cosmesis to top surface of the attachment plate after assembling to pylon as shown using Thixofix adhesive (926204) or equivalent and shape to suit. Do not adhere plate or foam to foot shell.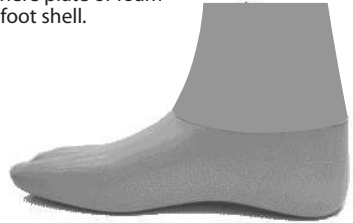


9 Dis-assembly

Ease the cosmesis attachment plate from the foot shell to gain access to the pyramid adjustment screws for adjustment or dis-assembly. Dis-assembly is the reverse of assembly.

10 Technical Data

Operating and Storage Temperature Range:	-15°C to 50°C (5°F to 122°F)
Component Weight [size 26]:	317g (11½ozs)
Recommended Activity Level:	3
Maximum User Weight:	125kg (275lb)
Proximal Alignment attachment:	Male Pyramid (Endolite) +/- 7°
Build Height:	[sizes 22-24] 65mm
[See diagram below]	[sizes 25-26] 70mm
	[sizes 27-30] 75mm
Heel Height	10mm

Fitting Length

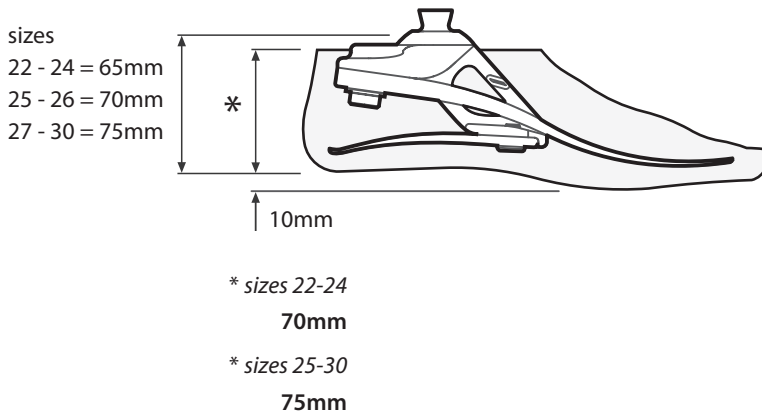11 Ordering Information:

Spring Kits				
Rate	Foot sizes			
	22-24	25-26	27-28	29-30
Set 1	539801	539810	539819	539828
Set 2	539802	539811	539820	539829
Set 3	539803	539812	539821	539830
Set 4	539804	539813	539822	539831
Set 5	539805	539814	539823	539832
Set 6	539806	539815	539824	539833
Set 7	--	539816	539825	539834
Set 8	--	539817	539826	539835

Foot shell (for dark add 'D')			
Small	Medium	Large	Extra Large
22L 539038	25L 539044	27L 539048	29L 539052
22R 539039	25R 539045	27R 539049	29R 539053
23L 539040	26L 539046	28L 539050	30L 539054
23R 539041	26R 539047	28R 539051	30R 539055
24L 539042	--	--	--
24R 539043	--	--	--

Glide Sock

Size 22 to 26 Part No. 531011

Size 27 to 30 Part No. 532811

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer.

CE Conformity

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

Warranty

The Esprit is warranted for 36months - foot shell 12 months - glide sock 3 months from original date of purchase unless otherwise stated.

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Endolite Deutschland GmbH
endolite India Ltd